Installing IMHome Universe 3.1 on Windows 10

The following instructions are a step-by-step guide to installing IM Home Universe on a Windows 10 computer. Depending on your security settings, you may not see the first few screens. If that is the case, please skip to Step 3.


Step 1: Please select the "More info" option on the Microsoft Defender SmartScreen window.



Step 2: Choose the option to "Run anyway" on the Microsoft Defender SmartScreen window.




Step 3: Select "Yes" in the User Account Control window to permit the installation.

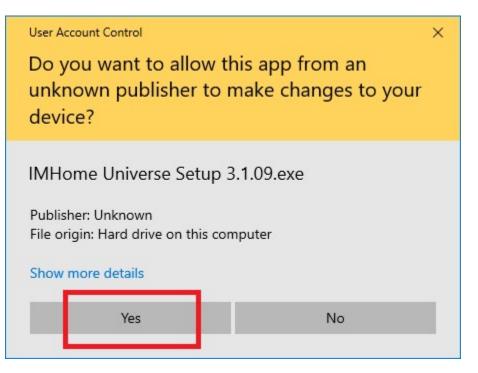

Step 4: Read and accept the Licensing Agreement and select "Next"

etup - IMHome Universe 3.1		
icense Agreement		
Please read the following important information before continuing.		(
Please read the following License Agreement. You must accept the agreement before continuing with the installation.	terms of t	this
Interactive Metronome Professional Licensing Agreement		^
I. DEFINITIONS for the purpose of this license:		
 a) LICENSOR means Interactive Metronome, Inc, a corp a principle place of business at 13798 NW 4th St., Suite 300 Sunrise 		
 b) PROFESSIONAL means you in your capacity as an ind clinical trainer of the Interactive Metronome products. 	vidual qu	alified
c) LICENSED PRODUCT means a MASTER CONTROL UNI	т ("мси")), ~
I accept the agreement		
○ I do not accept the agreement		
Nex		Can

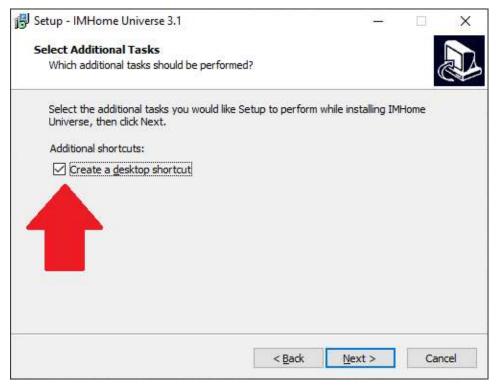

Step 5: Select the "Create a desktop shortcut" option and select "Next "

Step 6: Review all the settings and select "Install" to begin installing IM Home.

